

# Pre-Tension®

Preloaded Capsular Tension Ring  
in a single use injector



**NEW**



*Selfloading*

*Onehand-Technique*

*Implantation – clockwise or counter clockwise*

*Patented Hockey design of CTR*


*Implantation with 2.0 mm incision*

**OPHTHALMO**

*Pro*

GmbH

Products for ophthalmic surgery

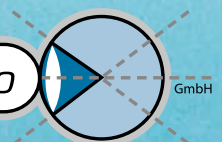


# Pre-Tension®

## Preloaded Capsular Tension Ring in a single use injector


Push the pushrod forward until it is limited.



Slowly put the Capsular Tension Ring into the tip of Injector.

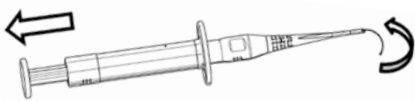

The Capsular Tension Ring is completely put into the tip of Injector.

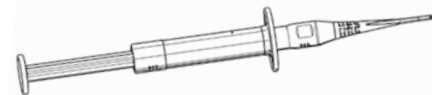

### Technical Data

Model	<b>Pre-Tension® 13/11</b>
Compression	From 13 mm to 11 mm
Material	Clear PMMA, flexible ring
Preloaded	Self-loading, single packaging a valuable addition to your surgical armamentarium
Injector	<ul style="list-style-type: none"><li>• Easy to use</li><li>• Total control (includes action/reaction mechanism)</li><li>• Implantation clockwise or counter clockwise (indicators on injector)</li><li>• Long small tip (especially convenient in cases of deep set eyes)</li><li>• Beveled tip (for easy entry into the incision)</li></ul>
Capsular Tension Ring	<ul style="list-style-type: none"><li>• Circular expansion and stabilization of the capsular bag</li><li>• Safe IOL centration in eyes with zonular dehiscence</li><li>• Prevents IOL decentration after capsular shrinking</li><li>• Stabilized conditions during Phaco-emulsification surgery</li><li>• Reduced risk of capsular fibrosis</li><li>• Improves visual acuity when implanted along with premium IOL</li></ul>

### Contact:

OPHTHALMO Pro GmbH  
Im Reihersbruch 1, 66386 Sankt Ingbert  
Owner: Thomas Zimmer  
Tel.: +49 (68 94) 99 88 770  
Fax: +49 (68 94) 99 88 777  
Email: [office@ophthalmo-pro.de](mailto:office@ophthalmo-pro.de)  
[www.ophthalmo-pro.de/en](http://www.ophthalmo-pro.de/en)



Products for ophthalmic surgery